

EN

**INSTRUCTION FOR USE**  
**SMART PREP**  
**Liquid 100ML**

**COMPOSITION**

17% Disodium EDTA, 0.7% Cetrimide, Excipient q s ad 100ml

**PROPERTIES**

Main component is EDTA, after injection into dentine forms a complex with dentine calcium, loosens the structure of hard tooth tissues and makes easier an enlargement of calcified and sclerous orifice of the canals, cleans and forms hard accessible tooth canals.

**INDICATIONS**

- Irrigation and Chelating agent in Root Canal Treatment
- Detection of root canal orifice
- Chemical enlargement of root canals during preparing for filling

**CONTRAINDICATIONS**

Drug idiosyncrasy  
Use with great care for allergic patient

**SIDE EFFECTS**

No known side effects as far as the application of the product is concerned

**METHOD OF USE**

Liquid is applied with the help of syringe from luer lock of bottle, with side vent root needle, cotton swab. Put it into the pulp cavity to start mechanical enlargement.

Then using root needle apply the liquid into root canal for 2-3 minutes, repeat the procedure alternating chemical, mechanical enlargement with impregnation of dentine with fresh portion of the liquid if necessary.

**PRESENTATION**

Liquid (Amber Bottle)                      100ml

**STORAGE**

Store at 5 °C to 25 °C, in a well sealed container Shelf life – 3 years

***Attention! Close Cap of bottle tightly after use***

The material must not be used after the expiry date

**MANUFACTURER**

SafeEndo Dental India Pvt Ltd  
A/6/2, Kamdhenu Industrial Estate  
Opp. Gorwa Water tank, Gorwa, Vadodara-390016  
E-mail: info@safeendo.org  
www.safeendo.org

***Not for Medicine use, For use in Dentistry by Dental Professional***

002-03-100

Date of last revision: 2020-11

**SafeEndo**  
enhance your endo

 +91 79-22139977 / 22163335  [www.safeendo.org](http://www.safeendo.org)  [info@safeendo.org](mailto:info@safeendo.org)  
[www.safeendoindia.com](http://www.safeendoindia.com)

**SafeEndo**  
enhance your endo

 +91 79-22139977 / 22163335  [www.safeendo.org](http://www.safeendo.org)  [info@safeendo.org](mailto:info@safeendo.org)  
[www.safeendoindia.com](http://www.safeendoindia.com)