

INSTRUCTION FOR USE

EXPLORE

COMPOSITION

Acid Rhodamine Dye in semi-aqueous base

PROPERTIES

Caries Detector Dye

Fast and Easy Use, Preserves Healthy Tooth Hard Substance, Stains Tooth Hard Structure RED changed by Dental Caries during all Preparatory measure in the cavity .

Dental product is produced for using in aggressive biological medium with the temperature range between 32°C and 42°C (in oral cavity at the temperature (37±1)°C and humidity 60-90 %).

INDICATIONS

Caries Disclosing Agent for Conservative Dentistry

CONTRAINDICATIONS

Drug idiosyncrasy

Use with great care for allergic patient

Don't misuse

SIDE EFFECTS

No known side effects as far as all terms of application, transportation and storage are observed

METHOD OF USE

Small amount of liquid is applied or painted on infected tooth surface or dentin surface by micro applicator tips for 10 seconds. Wash the liquid off with water and dry with compressed air. At the same time, it makes sure that as little

healthy dentin which can be remineralised is removed as possible, secures pulp vitality, and preserves the healthy tooth structure. Numerous studies have shown that it is not possible to distinguish, either optically or mechanically using the sensor test, between irreversibly diseased Dentin and healthy Dentin which can be remineralised. Using Explore, you can confidently proceed with the excavation because only the irreversibly diseased dentin is stained red.

PRESENTATION

Explore liquid 15ml Ref no 002-07-015

STORAGE

Store at 5 °C to 25 °C, in a well sealed container

Attention! Close materials tightly after use

The material must not be used after the expiry date

Shelf life – 3 years

MANUFACTURER

SafeEndo Dental India Pvt Ltd

A/6/2, Kamdhenu Industrial Estate

Opp. Gorwa Water tank, Gorwa, Vadodara-390016

E-mail: info@safeendo.org

www.safeendo.org

Not For Medicinal use, For use in Dentistry By Dental Professional only

002-07-015

Date of last revision: 2021-01

SafeEndo
enhance your endo



+91 79-22139977 / 22163335



www.safeendo.org
www.safeendoindia.com



info@safeendo.org

SafeEndo
enhance your endo



+91 79-22139977 / 22163335



www.safeendo.org
www.safeendoindia.com



info@safeendo.org