# INSTRUCTION FOR USE SmartTemp 40a

**Eugenol Free Temporary Filling Material** 

#### COMPOSITION

Zinc Oxide, Calcium Sulphate, Poly Vinyl Acetate Resin, Flavours and additives

SmartTemp is a paste consistency, adaptive in nature temporary filling material which hardens upon contact of moisture in Oral Cavity. Marginal seal of material is excellent, so it makes tight seals and closure of working area.

# Properties and Method of Application

SmartTemp is inserted with small portion with any suitable filling instrument in to cavity, due to its excellent mould ability it spreads in to cavity with gentle pressure to adapt material with cavity wall. We recommended in to wet cavity for better adaptation. Remove excess material by wiping it off with a moist cotton pellet towards margin of the cavity. It hardens under moisture, removal of material is so ease, you can remove material with use of diamond rotary burs or sharp filling instrument.

## INDICATION

As a temporary filling material for Root Canal Procedure and Cavity Restoration and Inlays and implant abutment sealer.

## CONTRAINDICATION

Drug idiosyncrasy

Use with carefully if person is allergic to PVC resin

Do not use directly on exposed Pulp area as a temporary or permanent closure









#### SIDE FEFFCTS

No any known side effects as far as all terms of application, transportation and storage are observed

## **PRESENTATION**

SmartTemp 40g iar Ref No. 005-03-040

#### STORAGE

Store at 5 °C to 25 °C, in a well sealed container Attention! Close materials tightly after use

The material must not be used after the expiry date Shelf life - 2.5 years

#### **MANUFACTURER**

SafeEndo Dental India Pvt I td

A/6/2, Kamdhenu Industrial Estate,

Opp. Gorwa Water tank, Gorwa, Vadodara-390016

E-mail: info@safeendo.org

| www.safeendo.org

Not For Medicinal use, For use in Dentistry By Dental Professional only

005-01-250 Date of last revision : 2021-03







